





**THE PEARLE STUDY** The PEARLE study was a **prospective, multi-center, pivotal, single-arm study** designed to assess the safety and effectiveness of the Jada System in treating primary postpartum hemorrhage and abnormal postpartum uterine bleeding, and enrolled 107 subjects.<sup>1</sup>

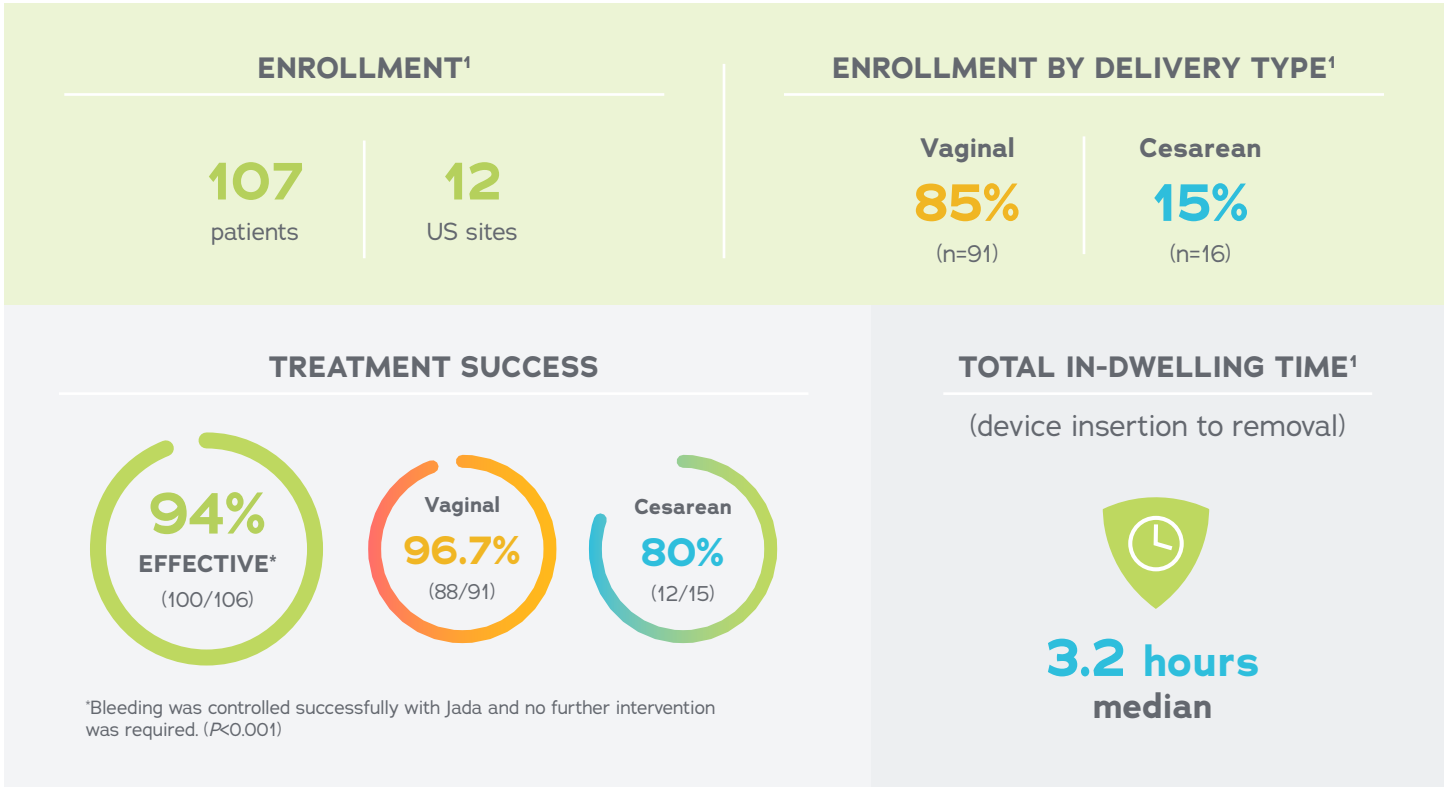

**SAFETY/ADVERSE EVENTS (AEs) RELATED TO THE JADA SYSTEM<sup>2</sup>**

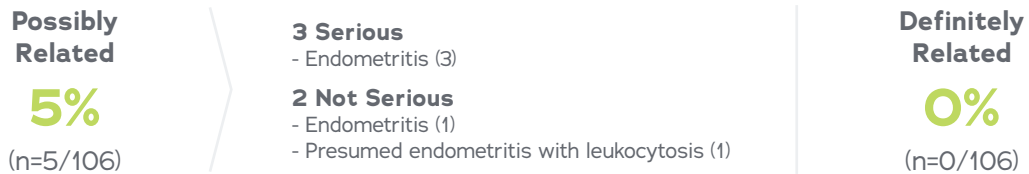

Please refer to the Jada System Instructions for Use for indications, contraindications, warnings, precautions, and other information at [thejadasystem.com/ifu](http://thejadasystem.com/ifu).


**References:** 1. D'Alton ME, Rood KM, Smid MC, et al. Intrauterine vacuum-induced hemorrhage-control device for rapid treatment of postpartum hemorrhage. *Obstet Gynecol.* 2020;136(5):882-891. doi:10.1097/AOG.0000000000004138 2. Data available on request from Organon Professional Services-DAP (Marketing Operations), 30 Hudson St., Jersey City, NJ 07302. Please specify information package US-JDA-110046.



The Jada® System is intended to provide control and treatment of abnormal postpartum uterine bleeding or hemorrhage when conservative management is warranted.



**THE RUBY STUDY<sup>1</sup>** is a **multi-center, observational, post-market registry** designed to collect observational data on patients treated with the Jada System in the post-market setting.

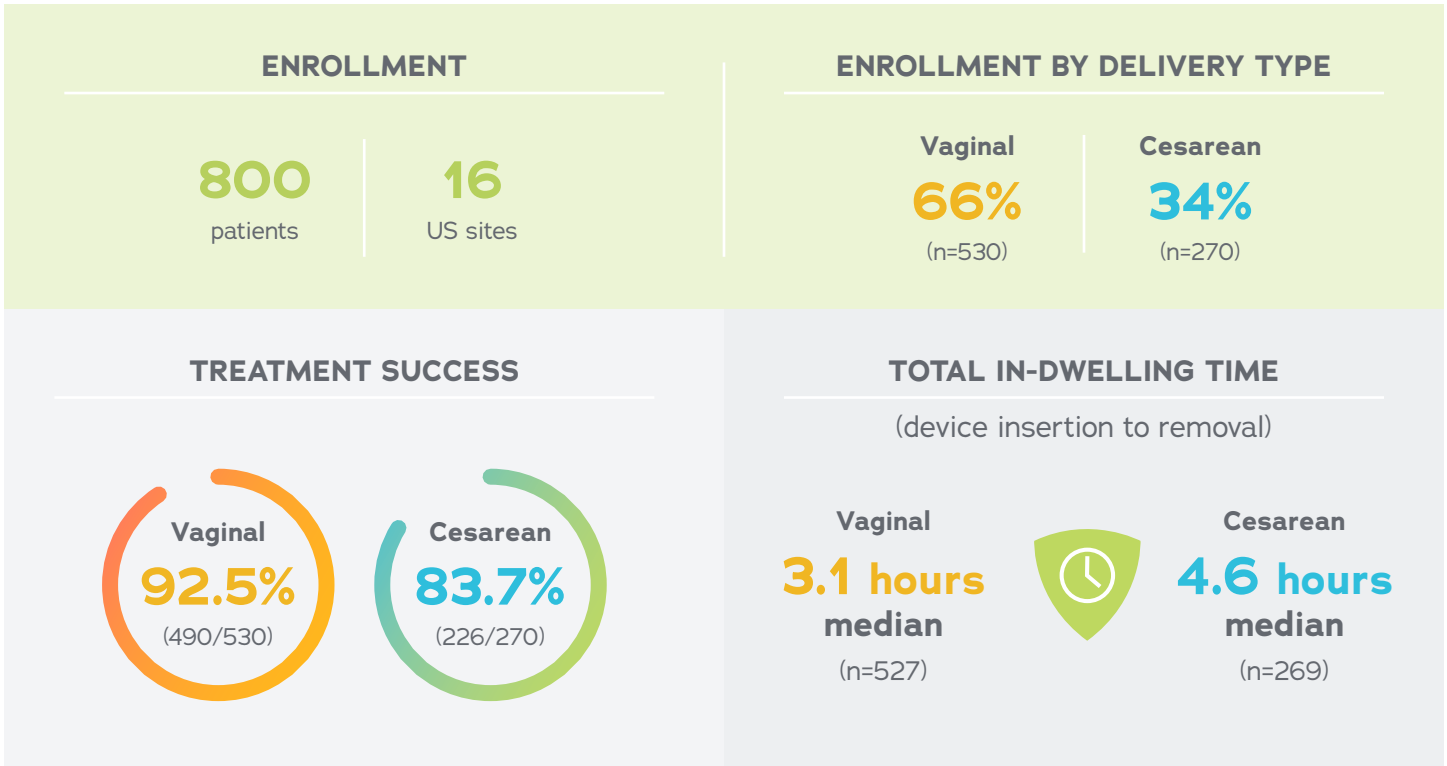

**SAFETY/ADVERSE EVENTS (AEs) RELATED TO THE JADA SYSTEM**

**Possibly Related**

**1.1%**

(n=9/800)

**3 Serious**

- Hemorrhagic shock & hypotension (1)
- Endometritis (2)

**6 Not Serious**

- Endometritis (3)
- Abdominal pain (1)
- Diarrhea/Emesis (1)
- Fever (1)

**Definitely Related**

**0%**

(n=0/800)

Please refer to the Jada System Instructions for Use for indications, contraindications, warnings, precautions, and other information at [thejadasystem.com/ifu](http://thejadasystem.com/ifu).

**Reference: 1.** Data available on request from Organon Professional Services-DAP (Marketing Operations), 30 Hudson St., Jersey City, NJ 07302. Please specify information package US-JDA-110427.

